



smileyscope™
reimagine healthcare

**Quick
User
Guide**

Quick tips to get started with Smileyscope

How do I use Smileyscope?

1

Set up the VR experience

- Wipe down the device and remove faceplate from headset
- Hold down the red button to turn on the device
- Use the touchscreen to select the appropriate program, click 'Get Started'
- Customize the program to best suit user requirements
- Reconnect faceplate to the headset

2

Introduce headset to patient

- The user will go on a 'VR adventure' during their procedure. The clinician and the patient should discuss the procedure and how the VR headset is going to work

3

Triple-click to start


- The user should be in a comfortable position. Place the headset on their head
- Press the green button 3 times to start the adventure

4

Wipe down and recharge

- Wipe down between each user¹
- Recharge device between uses

¹ Hospital grade antimicrobial wipes, including isopropyl alcohol, chlorhexidine, quaternary ammonium, hydrogen peroxide can be used to clean the device.

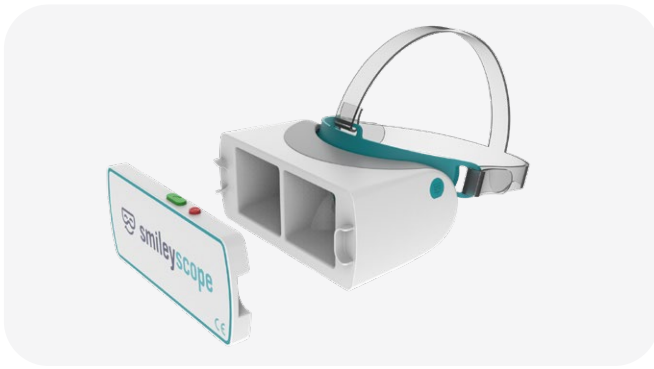


*Headset with
faceplate removed*



*Program
Selection Screen*



*Experience
Selection Screen*

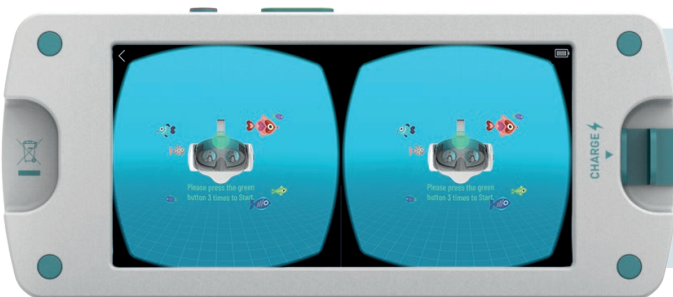

Calibration screen

Ensure the user is in a comfortable position with the headset on before pressing the green button 3 times.

Frequently Asked Questions

What ages is Smileyscope for?

Smileyscope has been clinically tested and is suitable for ages 4+ years.¹

Is Smileyscope infection control compatible?

Yes, the Smileyscope device has streamlined hardware with Advanced Anti-microbial Impregnation. It is designed to meet biological standard ISO 10993-1:2018.

Is Smileyscope clinically tested?

Yes, Smileyscope has been shown in two randomized clinical trials to reduce pain, anxiety and the need for restraints during pediatric needle procedures.¹

- ↓ 60% child pain
- ↓ 40% child anxiety
- ↓ 48% restraints

Is Smileyscope suitable for users with glasses?

Yes, the user can use Smileyscope with their glasses on. They should place the headset directly onto their face and then slip on the headband.

How long does the battery last?

The battery will provide 3-4 hours of continuous use. The device will go to sleep after 30 secs of inactivity.

What can I use to clean the device?

Hospital grade antimicrobial wipes, including isopropyl alcohol, chlorhexidine, quaternary ammonium, hydrogen peroxide can be used to clean the device.

Does a user need to be sitting upright?

No, Smileyscope will work for any user orientation. The user should be comfortable and in the desired position before pressing the green button 3 times.

The user cannot see VR experience in front of them. How do I fix that?

The green button should only be pressed 3 times when the faceplate is connected to the headset and on the user's head. This will ensure the VR experience is in the direction they are facing.

Does Smileyscope have training videos?

Yes, there are videos to show you how to use Smileyscope that can be viewed at our website and YouTube Channel: <https://bit.ly/SSlearn>

Is Smileyscope not recommended for any users?

Smileyscope should not be used by users with any of the following conditions:

- Neuromuscular compromise
- Epilepsy
- Contact infection precautions
- Skin hypersensitivity
- Unstable/deteriorating condition
- Poor skin integrity

For questions or technical issues please contact the following:

USA

✉ sales@smileyscope.com

☎ +1 888 300 7117

Australia & Rest of World

✉ info@smileyscope.com

☎ +61 1800 870 396

¹ Chan et al. (2019) Virtual Reality for pediatric procedural pain, The J of Peds